

# CUSTOMER INFORMATION BULLETIN

## NuSurgix Pro-Lux™ Proctoscope & Sigmoidoscopes



### Dear Customer,

As an environmentally responsible company, Rocket Medical plc. provides its customers with the choice to remove the battery cells from our Pro-Lux range of self-illuminating endoscopes prior to disposal, should they wish.

We provide Instructions for Use with the product to enable our customers to complete this procedure should they need to comply with local hospital or NHS Trust policies which specifically require the removal of battery cells prior to disposal.

**However, there is no requirement to do so. The device can be disposed of in normal clinical waste and incinerated following local hospital policies without removal of the battery cells.**

The small coin type battery cells contained in our Pro-Lux devices are lithium based and can be safely incinerated in a commercial hospital incinerator.

Prior to disposal the Pro-Lux Sigmoidoscope should be switched off using the button on the handle and placed into normal clinical waste.



The Pro-Lux Proctoscope employs a single cool light LED which cannot be switched off following activation. The proctoscope can be safely placed into clinical waste and sent for incineration with the LED illuminated.



Customer Information Bulletins on all of our products can be found on our web site at:  
[http://www.rocketmedical.com/Rocket\\_support\\_CIB\\_main.htm](http://www.rocketmedical.com/Rocket_support_CIB_main.htm)

**If you have any questions, please contact the Customer Services Team for assistance:**

Tel: +44 191 419 6988. Fax: +44 419 6889. Email: [customerservices@rocketmedical.com](mailto:customerservices@rocketmedical.com)

