

CUSTOMER INFORMATION BULLETIN

Rocket R54543 Chest Drain Unit

Dear Customer,

As part of our programme of continuous improvement we are pleased to introduce changes to the R54543 Triple Chamber Chest Drain.


- The suction control is now a dry suction control system. This means that the thoracic vacuum can be connected to the chest drain unit and the amount of suction can be controlled by turning a dial rather than the level of water.
- The unit works as single drainage unit, however if required, the device can be used as a dual drainage unit. The supplied extra tubing can be connected to a second chest catheter so that an apical and basal chest tube can be connected to the same chest drain unit.
- At present this product is being supplied by Rocket Medical on a distribution only basis, this means the label will carry the legal manufacturers details which includes but is not limited to catalogue, lot and expiry date information.
- Rocket Medical will add a label to each pack and outer carton detailing catalogue number (R54543) and lot details only.

Customer Information Bulletins on all of our products can be found on our web site at:

sales.rocketmedical.com

If you have any questions, please contact the Customer Services Team for assistance:

Tel: +44 191 419 6988. Fax: +44 419 6889. Email: customerservices@rocketmedical.com

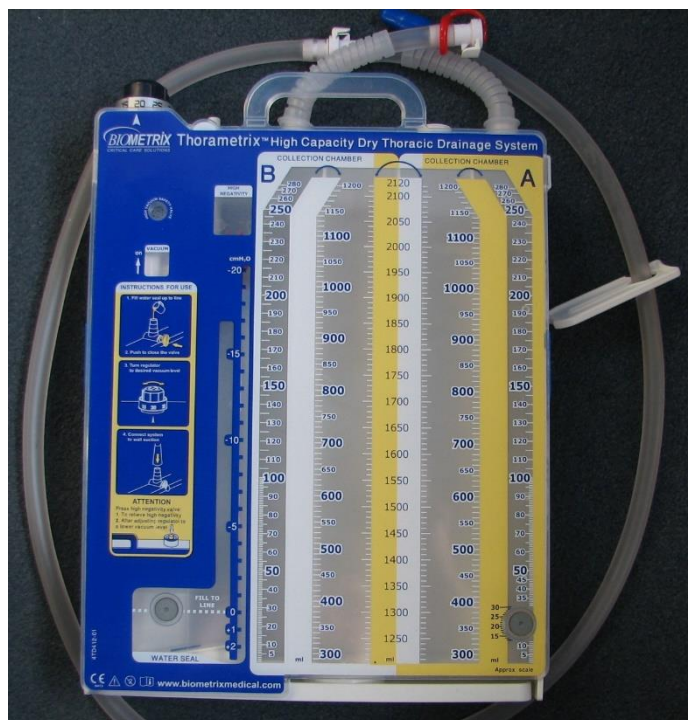

Rocket Medical plc, Sedling Road Washington Tyne & Wear NE38 9BZ England
T: +44 (0) 191 419 6988 **F:** +44 (0) 191 419 6889 **E:** customerservices@rocketmedical.com www.rocketmedical.com

Registered Office: Rocket Medical plc, Imperial Way, Watford, WD24 4XX, England Company No: 03276608

Registered office: Rocket Medical plc Imperial Way Watford WD24 4XX England. Company Number: 03276608