

CUSTOMER INFORMATION BULLETIN

Rocket ProLux™ Sigmoidoscope and Proctoscope battery storage.

Dear Customer,

As an environmentally responsible company, Rocket Medical plc. provides its customers with the choice to remove the battery cells from our Pro-Lux™ range of self-illuminating endoscopes prior to disposal, should they wish to do so.


We provide Instructions for Use with the product to enable our customers to complete this procedure should they need to comply with local hospital or NHS Trust policies which specifically require the removal of battery cells prior to disposal. However, there is no EU or national regulatory requirement to do so and it is quite safe to dispose of all Pro-Lux 'scopes complete with batteries.

Trays for the storage and disposal of batteries are provided in each outer case of sigmoidoscopes or proctoscopes. The small coin type battery cells contained in our Pro-Lux™ range are lithium based and can be safely incinerated in a commercial hospital incinerator.

Following the Instructions for Use, remove the battery pack and place one pair into each cell of the tray. Replace the lid supplied as the tray is filled.

IMPORTANT: Lithium cells are high powered batteries and can over-heat when short circuited.

To avoid the risk of multiple cells coming into contact with each other and causing a short-circuit, place the lid onto the tray when filled and tape securely into position at the mid point of each side.

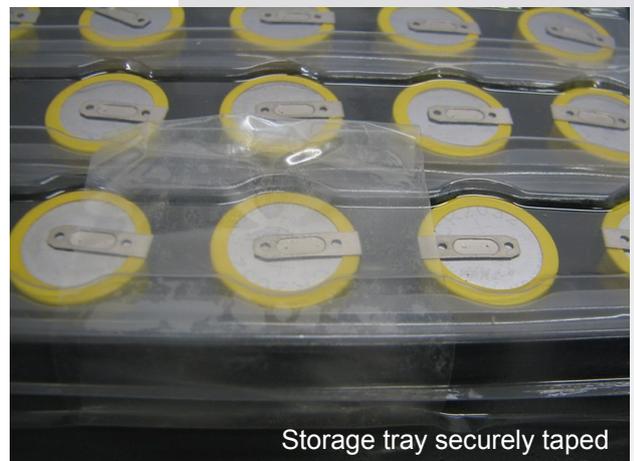

The use of tape to secure the tray lid is a temporary measure only. Rocket Medical are currently developing a more rigid storage and disposal tray to reduce this risk further.

Customer Information Bulletins on all of our products can be found on our web site at: www.rocketmedical.com

If you have any questions, please contact the Customer Services Team for assistance:

Tel: +44 191 419 6988. Email: customerservices@rocketmedical.com