

Chest Drainage System

10 EASY STEPS TO REPLACE A USED BOTTLE

REPLACEMENT BOTTLE SETUP



- 1 Rotate red sealing cap to open.
- 2 Fill to prime level with sterile water or normal saline, approx 500ml.
- 3 Clamp patient catheter close to the chest wall.
- 4 To remove the tube set from the used bottle, rotate to 'open' position and remove.

- 5 Insert tubing set into new bottle and twist Clockwise until 'click' is heard.
- 6 Unclamp catheter.



To seal

- 9 Tear off small red cap and place into vent/suction port.
- 10 Place red sealing cap into aperture and twist clockwise until 'click' is heard.



If suction is used

- 7 Remove green vent cap.
- 8 Connect suction tubing to vent/suction port.

Used bottle is now ready for disposal. According to hospital procedure.

RE-ORDER CODES

Single Chamber		Floor Stand	R54503
Blue Bottle	R54500	Double Bottle	
		Floor Stand	R54505
Tubing Set		Dual Collection Bottle	
-with connector	R54502	Green Bottle	R54509
-without connector	R54539		

USED BOTTLE DISPOSAL

