Rocket Copeland® Fetal Scalp Electrode

(GB)

INDICATIONS: A CTG electrode for attachment to the fetal presenting part to enable fetal heart rate monitoring. This device is only for use by or under the supervision of trained personnel and in accordance with national guidelines.

CONTRAINDICATIONS: Not to be applied the fetal face, fontanelle or genitals or in the presence of vaginal infection, HIV, infectious hepatitis, active herpes simplex virus. Known or suspected fetal coagulation disorder. Chronic fetal bleeding (e.g., due to partial placental abruption) leading to loss of fetal blood volume may result in a reduction in the margin of safety that the fetus can successfully respond to hypoxia.

1. PREPARATION

Conduct a vaginal examination and advance the Rocket® Copeland® between the examining fingers until the needle (A) is held pushed against the fetus by finger pressure on the serrated depression (B). Ideally apply to the scalp, avoiding the fontanelle. May also be placed on buttocks, legs or arms.

2. APPLICATION

With the fingers of the other hand, rotate (C) anti-clockwise and then POSITIVELY CLOCKWISE to insert the needle into the fetal scalp. Do not rely on spring tension to insert. The lead will have either two bare ends or a connector, which fit into the connecting parts of the fetal monitor. Follow manufacturers instructions for methods of attachment

3. REMOVAL

To remove turn (C) ANTI-CLOCKWISE. It is not necessary to hold (A).

4. WARNINGS AND PRECAUTIONS

In the event of a poor trace or no trace at all, follow this sequence until a trace is established.

1. Check that you have followed the instructions and applied the electrode correctly.
2. Check that the fetal monitor and paper recorder is switched on.
3. Check that the maternal legplate is correctly applied and connected to the monitor combination.
4. Check that the leadwires are securely inserted into the legplate, if necessary re-wire the leads.
5. If still no or poor trace, re-position the electrode.
6. If still no or poor trace, replace the electrode.

CAUTION: Do not use if packaging is damaged or opened.

NATURAL
LATEX

This device is not manufactured with
natural rubber latex

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