Rocket Indwelling Peritoneal Catheter Drainage Pack and Bag Set

Scope: These instructions cover code R54401 and derivatives of Rocket Indwelling Peritoneal Drain Pack and Bag Set.

Indications: For procedure preparation, bag attachment and drainage of abdominal ascites by patients fitted with Rocket Indwelling Drainage Catheter.

Contraindications: For single use with Rocket Indwelling Catheter only. Only drain as directed by your Doctor or Nurse.

Preparation:

1. Use a clean, clear workspace, ideally a table or bench top.
Important: Wash your hands with warm soapy water for at least 1 minute before you begin.
2. Remove the dressings from around the catheter site, if you notice anything unusual such as pain, redness, swelling or free fluid, finish this procedure first and then contact your doctor or nurse immediately.
3. Open your procedure pack, removing the inner blue package and position the blue wrap on your clean workspace with the tape facing upwards. Carefully remove tape and unfold the blue wrapping making sure the contents remain in the middle of the sterile wrapping.
4. Open the Drainage Bag pack and place the bag outside of the blue wrap. Ensure the access tip with sheath attached (arrowed) is placed onto the blue wrap. It is important the access tip remains sterile. **Figure 1.**
5. Put the gloves on. It is important that these remain clean. Pick the first one up by the inside of wrist fold and place on the opposite hand, repeat this for the other hand. If possible your bare hand should not come into contact with the outer surface of the glove.
6. Peel open the pouch containing the catheter cap and let it drop into the middle of the sterile wrap.
7. Remove access tip cover from the bottle line and place the tip onto the blue wrap.
8. Remove your catheter cap and gently clean around the outside of the catheter valve with an alcohol wipe.
9. Take the drainage line of the bag and gently insert the access tip into the valve. You may feel a little resistance while doing this, this is normal. Lock together as shown. **Figure's 2 & 3.**

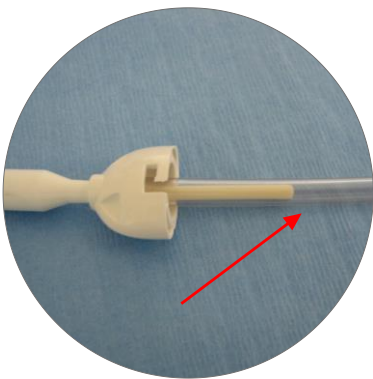
Fig.1

Fig.2

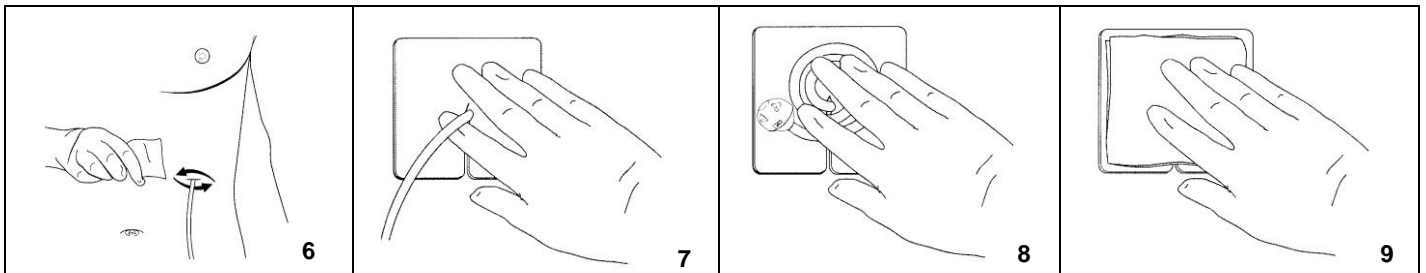
Fig.3

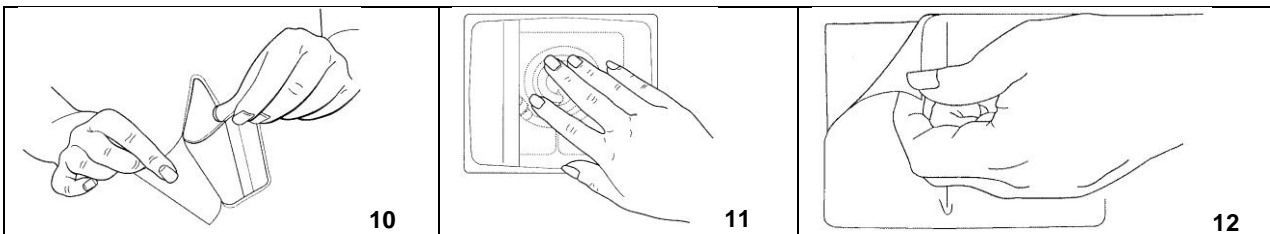
It is advised to sit down on a comfortable chair with the bag resting on the floor whilst draining fluid.

10. The drainage bag will now start to fill with fluid.
11. When the bag is full or you no longer continue to drain fluid, you have finished the drainage procedure. Carefully remove the drainage line from the catheter in a slow, smooth operation.
12. Use an alcohol wipe and gently clean around the outside of the valve. Take the new catheter valve cap and place onto the catheter valve – click into place.

13. Clean around the catheter exit site with an alcohol pad (**fig.6**).
14. Place the foam catheter pad around the catheter (**fig.7**).
15. Wind the catheter carefully into loops and place over the foam (**fig.8**). Cover the catheter with the gauze pads (**fig.9**).

16. Take the wound dressing and peel away the paper backing from one side (**fig.10**).
17. Position the dressing centrally over the gauze pads and carefully press down (**fig.11**). Then peel the other half of the dressing away pressing firmly all the time. Hold one of the coloured tabs on the dressing and pull the cover sheet away from the rest of the dressing, smooth and press until secure (**fig.12**).

IMPORTANT: These instructions are intended as a guide ONLY; please refer to the patient information literature and the training you will have received on the correct drainage procedure. If you are uncertain please contact your doctor or nurse.

CAUTION: Only drain as directed by your doctor. Do not change frequency or amount of fluid drained without consulting your doctor first.

WARNING:

During the drainage procedure it is quite normal that you may feel some abdominal discomfort. Contact your nurse or doctor if pain or discomfort are prolonged or severe. Pain may be an indication of infection. Do not use sharp objects such as scissors near the catheter.

IMPORTANT:

If you accidentally cut or pierce your catheter, use the blue slide clamp and immediately clamp off your catheter. Position your fingers between your body and the cut – pinch the catheter closed. Take the small slide clamp and slip it over the catheter then push the catheter into the small end of the clamp. This will close off the catheter preventing air entering or fluid leaking. (**fig.13**). Contact your doctor immediately.

13

This device is not manufactured with natural rubber latex

Rx ONLY

Disposal: This device should be handled and disposed of in accordance with local community policy and with regard to all applicable regulations, including but without limitation to, those pertaining to human health & safety and care of the environment.

DO NOT RESTERILISE

Unless opened or damaged, contents of package are sterile.
Store at room temperature. Avoid prolonged exposure to elevated temperatures.

Manufactured in the UK by: **ROCKET MEDICAL PLC**, Sedling Road, Washington, NE38 9BZ, England www.rocketmedical.com
Copyright© 2013. **ROCKET MEDICAL PLC** All Rights Reserved. Made in England.

0088

ZDOCK175 310817 Rev.10 Copyright© 2013-2017. **ROCKET MEDICAL PLC** All Rights Reserved.

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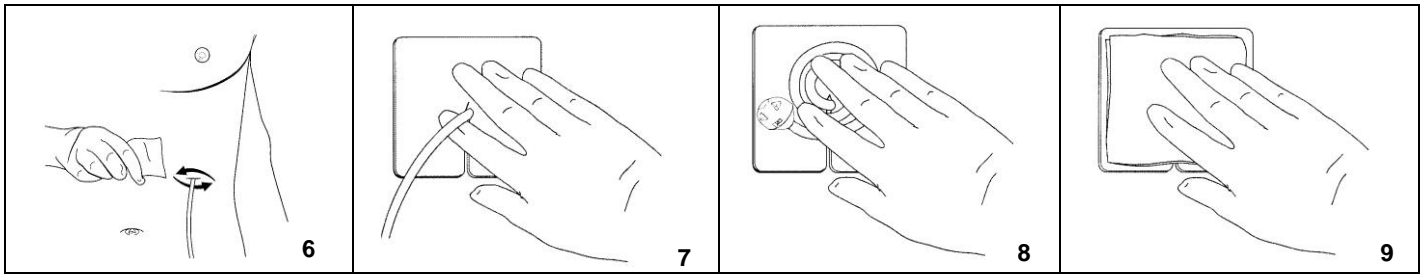
WAŻNE:

If you accidentally cut or pierce your catheter, use the blue slide clamp and immediately clamp off your catheter.

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R_X ONLY

Utylizacja: This device should be handled and disposed of in accordance with local hospital policy and with regard to all applicable regulations, including but without limitation to, those pertaining to human health & safety and care of the environment.

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ZDOCK175 310817 Ver. 10 Copyright© 2013-2017 **ROCKET MEDICAL PLC** Wszystkie prawa zastrzeżone. (PL)
Copyright© 2013. **ROCKET MEDICAL PLC** Alle Rettigheder Forbeholdes. Fremstillet i England
Copyright© 2013. ROCKET MEDICAL PLC Tous droits réservés. Fabriqué en Angleterre.
Copyright© 2013. **ROCKET MEDICAL PLC** Alle rechten voorbehouden. Gefabriceerd in Engeland.
Copyright © 2013. **ROCKET MEDICAL PLC** Alle Rechte vorbehalten. Made in England