Rocket R54410-00-VV IPC Valve Replacement Kit

INSTRUCTIONS FOR USE

Scope: These instructions cover the Rocket R54410-00-VV Valve Replacement Kit and derivatives.
Upon removal inspect the product to ensure that no damage has occurred.

Device Description: The Rocket R54410-00-VV Valve Replacement Kit contains replacement valve, clamp, scissors and dressing pack.

Indications

For replacement of damaged IPC valve

Contraindications

Not to be used with anything else but 16fg silicone IPC valve

This valve is designed to work with silicone catheters with an internal diameter of 2.7mm (8FG), suitability to be used with the chosen catheter is at the discretion of the clinician responsible for the patients care.

Procedure:

1. If a dressing is in place over the IPC catheter remove.
2. Using an aseptic technique open the valve replacement pack on to a clean and prepared surface.
3. Put on gloves.
4. Using the catheter clamp attach to the silicone catheter. Clamp shut to ensure no air or fluid can travel either way through the catheter.
5. Using the sterile scissors provided cut through the silicone catheter approx 1cm from the end of the valve
6. Discard old valve.
7. Push fit new valve, ensuring a good fit in the catheter.
8. If required proceed with drainage.
9. Redress catheter as per usual instructions.

This Device is not manufactured with natural rubber latex

SINGLE USE ONLY

Disposal: This device should be handled and disposed of in accordance with local hospital policy and with regard to all applicable regulations, including but without limitation to, those pertaining to human health & safety and care of the environment.

WARNING: If catheter becomes damaged clamp using supplied slide clamp or catheter clamp near to the body and seek advice with regards to preventing further damage to the catheter.

Federal (USA) law restricts this device to sale, distribution and use by or on the order of a physician.

DO NOT RESTERILISE

CONTINUOUS USE SHOULD NOT EXCEED 28 DAYS.

Unless opened or damaged, contents of package are sterile.

Store at room temperature. Avoid prolonged exposure to elevated temperatures.

Manufactured in the UK by: **ROCKET MEDICAL PLC**, Washington, NE38 9BZ, England www.rocketmedical.com
Copyright© 2013. **ROCKET MEDICAL PLC** All Rights Reserved. Made in England.

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Copyright© 2010. **ROCKET MEDICAL PLC** Tous droits réservés. Fait au Royaume Uni.
Copyright© 2000. **ROCKET MEDICAL PLC** Alle Rettigheder Forbeholdes. Fremstillet i England