

Rocket Amnilume™ Single Use Light Source

INSTRUCTIONS FOR USE

These instructions apply to all R57004-NS Rocket Amnilume™ Single Use Light Sources and derivatives.

Indications: Self-contained, battery powered LED light source for attachment to Rocket NS coded 'new style' amnioscopes for illumination during fetal blood sampling. Sterile, for single use.

Contraindications: Not intended for use where fetal blood sampling or the insertion of an amnioscope is contraindicated.

Preparation:

- Using aseptic technique, open the **Rocket Amnilume™** pack, remove the device onto a sterile area. Remove the obturator from the amnioscope.
- The **Rocket® Amnilume™** locates into the housing on the lower rear edge of the amnioscope.
- Angle the Rocket® Amnilume™ slightly downwards and slide forward until it locates securely into the slot
- The action of pushing the **Amnilume™** into position will automatically activate the light.
- Re-fit the obturator prior to insertion.
- Follow local hospital procedures to prep the perineum and vagina using an approved antibacterial solution. Insert the lit amnioscope, complete with obturator, into the vagina up to the fetal presentation.
- Remove the obturator.
- Complete the sampling procedure and remove the amnioscope.
- To turn OFF the **Rocket® Amnilume™** remove it from the amnioscope by sliding the light module out of the housing. Retain in the sterile area for subsequent samplings.
- Repeat from Stage 3 for each sampling.



IMPORTANT

Ensure the device is turned OFF between samplings.
Duty cycle: 5 x 15min ON periods with 10mins OFF between each period.

WARNING: This device contains 2 lithium-based primary cells.

**Do NOT attempt to disassemble the device.
Do NOT attempt to recharge or short circuit the batteries.
Lithium cells are suitable for incineration or landfill disposal**

CAUTION: DO NOT USE ETHYL CHLORIDE SPRAY – this may adversely affect the internal surface of the plastic amnioscope

Disposal:

This device should be handled and disposed of in accordance with local hospital policy and with regard to all applicable regulations, including but without limitation to, those pertaining to human health & safety and care of the environment



This device is not manufactured with natural rubber latex



Rx ONLY

STERILE EO



Li-ion

EC REP

Rocket Medical GmbH
Am Rosengarten 48,
15566 Schöneiche.
Germany



ROCKET MEDICAL PLC Sedling Road, Washington,
England, NE38 9BZ www.rocketmedical.com

DO NOT RESTERILISE

Unless opened or damaged, contents of package are sterile.
Store at room temperature. Avoid prolonged exposure to elevated temperatures.

NOT FOR LONG TERM USAGE



0088

ZDOCK287

300719

Rev.02

Copyright© **ROCKET MEDICAL PLC.**

All Rights Reserved.

(GB)

“Uncontrolled Copy – subject to change at any time without notification”