BRUKSANVISNING

Hegenberger Retractor

A1390

Bruksanvisning.....	2
Instruction for use.....	3
Brugsanvisning.....	4
Brugsanvisning.....	5
Käyttöohjeet.....	6
Gebrauchsanweisung.....	7
Instructions d'utilisation.....	8
Gebruiksaanwijzing.....	9
Instrukcja użytkowania.....	10
Kasutusõpetus.....	11
Naudojimo instrukcijos.....	12
Lietošanas instrukcija.....	13
Istruzioni per l'uso.....	14
Οδηγίες χρήσης.....	15
instruções de uso.....	16
Instrucciones para el uso.....	17
Uputstvo za upotrebu.....	18
Návod k použití.....	19

sv

en

no

da

fi

de

fr

nl

pl

et

lv

lt

it

el

pt

es

sr

cs

Wing Plast AB

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

1. Open the pouch and remove the Hegenberger Retractor. Hold the Hegenberger Retractor as shown in the image.


2. Gently press the sides of the Hegenberger Retractor together before inserting. **Do not use gel or lubrication on the Hegenberger Retractor.**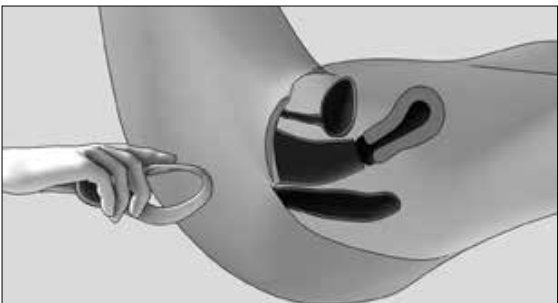


3. It is important that the patient be in a supine position with the legs elevated.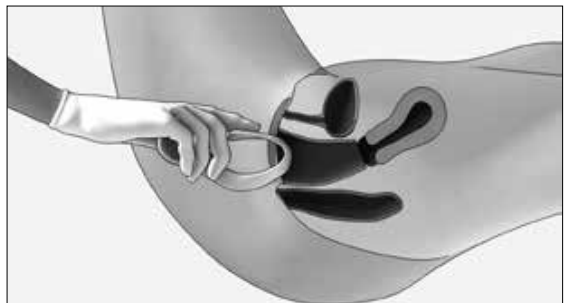


4. Keep the sides of the Hegenberger Retractor compressed while inserting it in the vagina. Follow the direction of the pelvic canal.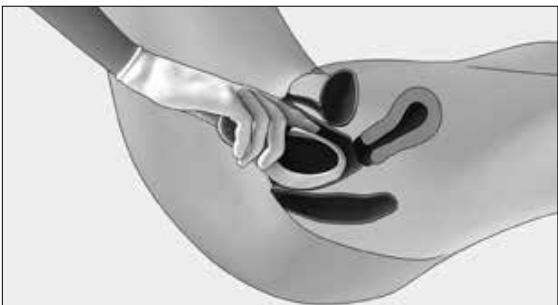

5. Place the Hegenberger Retractor all the way in so that the circles are resting on labia majora.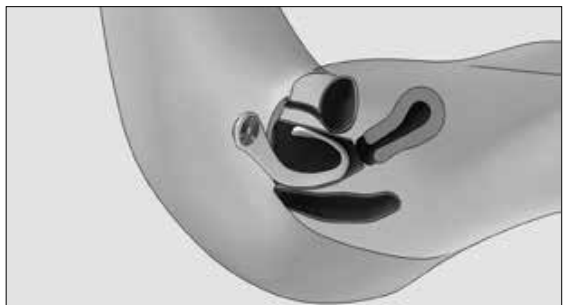

6. Correct placement is shown in this image.

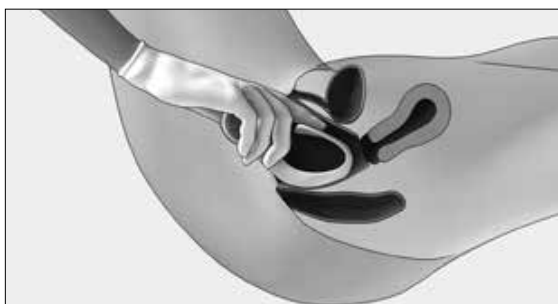Removing the Hegenberger Retractor

7. Gently press the sides of the Hegenberger Retractor together before removing. It is important that you use your index finger to ensure the Hegenberger Retractor is clear of the cervix.

Press down on the top part and gently retract the Hegenberger Retractor from the vagina.

Contraindications

The Hegenberger Retractor is only intended for use on women following vaginal delivery. It is only intended for use by trained clinical professionals after understanding the instructions for use.

The patient must be in a supine position with elevated legs.

Not to be used for pediatric patients or for placement in patients who have not delivered or in cases where force would be necessary for insertion.

Not to be used in cases involving post-partum haemorrhage.

Not to be used without informed consent from the patient.

Not to be used during pregnancy.

Not to be used for anterior wall procedures.

Not for reuse.

Explanation of labeling on labels and instructions for use.

 See Instructions for use	 Manufacturing day	 Don't reuse
REF Article number	 Expiration date	STERILE EO Sterilized with ethylene oxide
LOT Lot number	 Don't use if package is damaged	 Don't sterilize for reuse
MD Medical device	 Humidity 0-70%	 Temp range 0 to 30 ° C
 Keep dry	 Protect from sunlight	CE 0402 In accordance with the Medical Device Directive 93/42 / EEC (0402).

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Wing Plast AB, Nitgatan 11, 333 33 Smålandsstenar, Sweden, +46 371 330 30, www.wingplast.se