




- **EASIER SAMPLE COLLECTION:**

The sampling wand combines blade and capillary tube holders, to facilitate continuous visibility of the sampling site, which can be lost when transferring between separate devices.

- **SIMPLE, SAFE BLADE LOADING AND RETRACTION:**

The sampling wand features a retractable blade to reduce the potential for needlestick injuries. The pre-loaded FBS blade is advanced for the use by depressing the rear moulding. After the scalp incision, the blade is retracted by depressing the button on the wand shaft. Once retracted the capillary tube is ready for sampling.

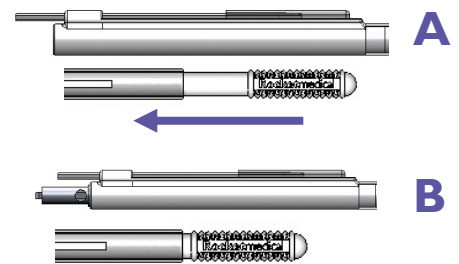


- **A RANGE OF CAPILLARY SIZES:**

Three kits are available, with capillary tube volumes of different sizes to meet the requirements of most blood gas analysers.

- **STERILE, FOR SINGLE PATIENT USE:**

Supplied in cartons of 10 wands.

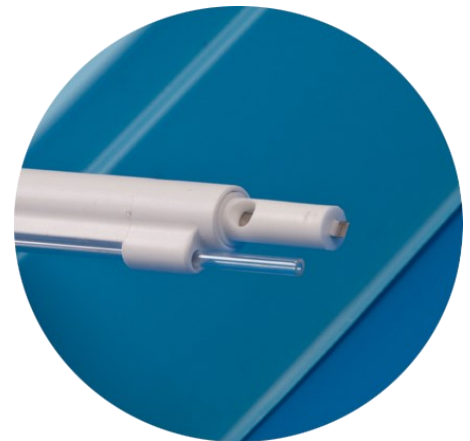

Description	Code	Tube Volume	FBS Kit Equivalents
<b>FBS Sampling Wand</b> Complete with 5 heparinised capillary sampling tubes. Sterile for single patient use. Supplied in cartons of 10 wands.	<b>R57021-55-SW</b>	55µl	R57018-00-SW
	<b>R57021-85-SW</b>	85µl	R57024-00-SW
	<b>R57021-10-SW</b>	100µl	R57026-00-SW