# FBS Sampling Wand

## INSTRUCTIONS FOR USE

**Kit Contents:** One sampling wand and five capillary tubes.

**Intended Use/Indications:** For the puncture of the fetal scalp and collection of fetal blood for the determination of fetal blood gas/pH during labour in cases of suspected fetal distress. This device should be used by or under the supervision of trained personnel and in conjunction with clinical practice guidelines. These instructions cover all R57021 series of FBS Sampling Wands, derivatives and kits.

**Contraindications:** This procedure is contraindicated with placenta previa, when identification of the presenting part is uncertain, in the presence of genital infections (e.g. herpes, Group B streptococcus, gonorrhoea, HIV/AIDS and hepatitis B), where the mother is a confirmed carrier of haemophilia and the fetus is either affected or of unknown status, if the woman is less than 34+0 weeks pregnant, or acute fetal compromise (for example, fetal bradycardia of >3 minutes).

**CAUTION: Avoid fetal face, fontanelles and genitalia.**

**CAUTION: DO NOT USE ETHYL CHLORIDE SPRAY – this may adversely affect the internal surface of the plastic amnioscope.**

**CAUTION: Failure to use correct blood volume select the correct analysis mode, the use of clotted or paraffin contaminated samples or the presence of air bubbles in the sample may lead to errors in results or rejection of the whole sample. Carefully follow manufacturer's instructions.**

### Capillary Tube Selection:


Rocket Medical supplies a number of FBS Sampling Wand kits with different capillary tubes. Please check your analyser user manual to confirm capillary tube compatibility. Where in doubt, please contact your analyser manufacturer representative.

Use the lowest volume compatible tube size to minimise the volume of blood required for collection

Product Code	Tube Volume / $\mu$ l	Tube Length	Tube OD	Tube ID	Heparin Type	Concentration /IU/ml
R57021-55-SW	55	59mm +2.0 - 0.0mm	2.05 $\pm$ 0.1mm	1.25 $\pm$ 0.05mm	Na	240
R57021-85-SW	85	87.0 $\pm$ 0.15mm	2.07 $\pm$ 0.06mm	1.17 $\pm$ 0.02mm	Na	240
R57021-10-SW	100	85.0mm	2.3 $\pm$ 0.05mm	1.20 $\pm$ 0.05	Ca/Na/Li	200

### Preparation:

- Using aseptic technique, remove the device from its outer bag.
- Select a capillary tube from the pack or kit and load into the holder. Locate the end of the tube into the holder at ❶ and then advance into the read tube grip at ❷. Ensure the tube is secure before sampling.
- Follow local hospital procedure to prep the perineum and vagina and complete a vaginal examination to establish the cervical dilatation and station of the fetal head.
- "Rocket Medical recommends the use of Amnilume™ Disposable Light Sources". Locate the Amnilume™ in the slot in the endoscope and push gently to engage. The light will activate when the light source is correctly fitted.
- Between samplings, remove the Amnilume™ light source from the endoscope by pushing off the with the thumb.
- Insert the endoscope, complete with obturator, into the vagina up to the fetal presentation. Ensure that the cervix is not trapped. Remove the obturator. Position the bevelled end to obtain the best possible seal against fetal presentation by rotating the endoscope. Maintain close contact between the amnioscope and the scalp to prevent leakage of liquor.
- Clean the puncture area using the green, x-ray detectable gauze swabs and forceps provided.
- With a clean swab stick, apply a *thin* layer of white soft paraffin (supplied) to the puncture area to promote capillary blood aggregation. **Remove excess with one or two dry swab sticks before performing the skin puncture.**
- The sampling wand comes pre-fitted with an FBS blade. Lightly holding the shaft and rear finger grip (A), push the grip forward to expose the blade (B). **Ensure that the blade is locked forward before use.**
- Perform the skin puncture using either technique ❶ make a vertical stab, rotate the blade 90° and make a second stab to form a cross or ❷ make a vertical stab, then angle the blade upwards and extend using the front edge of the blade to produce a single 4mm incision. Always wait for a good sized (3-4mm) bead to form on the surface of the paraffin before collection. Never slash at the scalp and avoid multiple small stabs
- To collect the blood sample, depress the button on the wand shaft to retract the FBS blade, and angle the capillary tube into the bead.
- Rapidly collect the blood sample, filling the tube completely, **without air bubbles**. Collection needs to be expedited to avoid blood clotting. **CAUTION: Do NOT use the sample if the blood clots.**
- Remove tube from holder and cap both ends. Take to the analyser immediately. Follow the analyser manufacturer's instructions as regards sample agitation or mixing. **CAUTION: Avoid distributing air through the sample; samples containing air bubbles may be rejected by the analyser.**
- Always ensure that the **clean**, non-paraffin exposed end is presented to the analyser. Note: some analysers require a 'clot catcher'.. Please refer to the manufacturer's instructions for guidance on when a clot catcher is needed.
- Always ensure that the **correct** measuring mode is selected *before* performing the analysis.
- For repeated samples commence at Step 7.

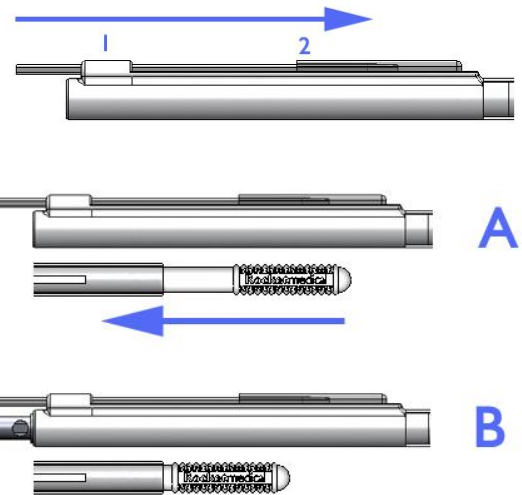### Post sampling:

- Haemostasis is accomplished by applying pressure to puncture site with a swab for one full contraction.
- If indicated, (refer to clinical guidelines) repeat procedure. The kit is for single patient use for the collection of up to 5 individual samples.

**Disposal:**

This device should be handled and disposed of in accordance with local hospital policy and with regard to all applicable regulations, including but without limitation to, those pertaining to human health & safety and care of the environment

This device is not  
manufactured  
with natural  
rubber latex

**STERILE EO****Rx ONLY****MD**

ROCKET MEDICAL PLC Sedling Road, Washington,  
England, NE38 9BZ [www.rocketmedical.com](http://www.rocketmedical.com)

**EC REP**

**Rocket Medical GmbH**  
Am Rosengarten 48,  
15566 Schöneiche.  
Germany

**For Single Patient Use Only.** Do not reprocess or re-sterilise as doing so may compromise the structural integrity of the device, leading to device failure; potentially the cause of serious harm to patients and users. Reuse on another person, reprocessing or re-sterilisation may also result in serious harm to patients and users from cross-contamination and infection with transmissible diseases.

Unless opened or damaged, contents of package are sterile. Store at room temperature. Avoid prolonged exposure to elevated temperatures.

**If the device is reprocessed and/or used across multiple patients there is risk of infection, damage or malfunction.**

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