Rocket Chest Drainage System

INSTRUCTIONS FOR USE

Indications: For drainage of air and/ or fluid from the pleural space and / or mediastinum.

Contraindications: Not intended for any other use except as indicated above.

Further Information: Rocket® Chest Drain Bottles have an antifoaming agent placed into them during the manufacturing process this can give the appearance of streaks or smears inside the bottle.

Antifoam significantly reduces foaming that can occur as air bubbles through the bottle contents. The antifoam agent is introduced into the bottle as a clear liquid, where it may coat the sides and base of the bottle before it dries.

INSTRUCTIONS FOR SET UP

1. Take one sterile bottle
2. Inspect outer bags - if ruptured do not use
3. Peel apart pouch and remove bottle.
4. Remove red bayonet sealing cap by rotating anti-clockwise from "CLOSE" to "OPEN" position.
5. Fill bottle with fluid to prime level noted on label.
6. Open tubing package and attach tubing set to bottle by inserting straw and cap into aperture and twisting clockwise until clicked shut. Take care not to touch the straw.
7. Once underwater seal has been established, place bottle on the floor and connect drainage catheter to tubing connector.
8. If required, insert bottle into Rocket Chest Drainage Bottle Stand

IF SUCTION IS USED

1. Attach suction pump tubing to the suction port.
2. Switch on suction pump and regulate to the required negative pressure.

IF SUCTION IS NOT USED

1. The **GREEN** cap marked **V** (for vent) is a venting cap, which allows the free flow of air from the bottle. **IT DOES NOT SEAL THE BOTTLE.**
2. Please ensure the **GREEN** vent cap is inserted into the suction port on the top of the bottle, as this acts as a dust cover.
3. **DO NOT** place the red cap over this port during drainage, as this is a sealing plug for disposal.

TO CHANGE THE BOTTLE

1. Obtain a replacement bottle and remove from packaging, having first checked for sterile integrity.
2. Remove red bayonet sealing cap and place onto bracket on lid of bottle.
3. Fill bottle with fluid to prime level noted on label.
4. In line with hospital policy clamp shut the tubing set in use.
Turn the bayonet closure cap on the tubing set anti-clockwise from the "CLOSE" position to "OPEN" position, and remove, taking care to shake off excess fluid in straw before removing from bottle.
5. Insert tubing set into replacement bottle and twist in a clockwise direction from "OPEN" position to "CLOSE" position
6. Replace bottle in stand, if required.
7. Unclamp tubing set.
8. Transfer suction pump tube to the replacement bottle, if suction is used.

DISPOSAL OF FULL BOTTLE AND TUBING SET

1. Remove tubing set for disposal in the usual manner by turning the clear plastic bayonet closure cap anti-clockwise from "CLOSE" position to "OPEN" position.
2. If suction has been applied, remove the suction pump tubing, tear/pull the **RED** seal cap located on the bottle lid and marked **SEAL**, from its tether.
3. **PUSH THE RED SEALING CAP FIRMLY WITH A TWISTING ACTION** into the suction port on the lid of the bottle to seal.
4. Remove the RED BAYONET cap from its bracket on the side of the lid and replace in hole vacated by the tubing set.
5. Seal cap in place by rotating the cap clockwise from "OPEN" to "CLOSE".
6. Should the red bayonet cap be mislaid, do not remove tubing set, but cut through the tubing and place the end of the tubing over the suction port on the lid of the bottle to create a seal.
7. **Place the adhesive label over the positive pressure blow off valve** 
8. Bottle is now ready for disposal.

WARNINGS AND CAUTIONS

1. Ensure that the positive pressure blow off valve and the green vent plug is not obstructed in any way.
2. Ensure that the drainage bottle is held in an upright position and below the patient's chest for maximum drainage efficiency and patient protection.
3. Routinely check patient tubes, tube connections, and Water seal compartments to verify system operation and tube patency.
4. This unit is a Sterile Single Use Device, sterilised by EtO, do not reuse.
5. If the Chest Drainage Bottle is damaged prior to or during use, replace it immediately with a new unit.
6. Caution: Only persons trained in Thoracic cavity drainage techniques should use this system.

GRADUATION SCALE - DEGREE OF ACCURACY +/- 7%

This device is not manufactured with natural rubber latex

ROCKET MEDICAL PLC Sedling Road, Washington, England, NE38 9BZ www.rocketmedical.com

Rocket Medical GmbH
Am Rosengarten 48,
15566 Schöneiche.
Germany

DO NOT RESTERILISE

Unless opened or damaged, contents of package are sterile. Sterile unless package has been opened or damaged
Store at room temperature. Avoid prolonged exposure to elevated temperature.

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Copyright© ROCKET MEDICAL PLC 2013-2020