

URIPOT™ URIKONE™




INSTRUCTIONS FOR USE

These instructions cover all URIPOT™ & URIKONE™ midstream urine collection devices.
This device is suitable for use by children under nursing or parental supervision.

1. **Wash your hands**, and remove wipe, if present, along with the bottle and funnel.
2. Use wipe to cleanse genital area. **Discard after use**
3. Attach funnel to bottle.
4. Pass a small quantity of urine into the toilet, then stop the flow.
5. Pass urine into the funnel until approximately half filled. Pass remainder of urine into the toilet.
6. Pour any excess urine in the funnel back into the toilet. **Do not allow the bottle to empty.**
7. Remove funnel by gently twisting away from the bottle. **Discard the funnel.**
8. Screw cap firmly onto the bottle to seal it. **Now wash your hands again.**
9. Return bottle and label as instructed.

ZDOCK127 Rev 06 All rights reserved. Copyright Rocket Medical plc2003-22

Disposal: This device should be handled and disposed of in accordance with local hospital policy and with regard to all applicable regulations, including but without limitation to, those pertaining to human health & safety and care of the environment.

 **ROCKET MEDICAL PLC** Sedling Road,
Washington. NE38 9BZ. England
www.rocketmedical.com

EC REP

Rocket Medical GmbH
Am Rosengarten 48,
15566 Schöneiche. Germany



This device is not
manufactured with
natural rubber latex

STERILE



Rx ONLY

MD UK
CA

CE
1639