# Rocket Ambulatory Chest Drainage Bag

## INSTRUCTIONS FOR USE

**Device Description:** Chest drainage bag for ambulatory patients, inpatient or outpatient use. Supplied sterile for single patient use.

**Indications:** For use as a collection device following chest cannulisation. Specifically, for those patients with increased mobility and who have a minimal air leak and/or drainage volume. If the patient has a large air leak open the blue valve.

**Contraindications:** Not intended for any other use except as indicated above.

**Pack Contents:** Ambulatory Chest Drain Bag with Brisk air leak valve, integral vent discs, activated carbon filter, flutter valve, and empty valve, fir-tree barb connector, Belt Strap, Priming Syringe, and Overnight Bag Connection

**CAUTION:** The drainage bag **MUST** be primed to ensure the internal one-way valve is working correctly before use

**Procedure:**

1. To prime the drain, use the 60ml syringe supplied, extended to a minimum of 50ml. Pass air through the drainage tube by inserting the syringe nozzle into the drainage tube and compressing the syringe. Air will pass through the flutter valve. This makes sure the valve has not been compromised whilst it has been in storage.
2. If required, fit the fir-tree barb connector provided, to the drainage tubing on the ambulatory bag so that it can connect to a plain ended chest catheter. (If the patient has a luer lock Seldinger type drain in situ this will not be required.)
3. If the patient has a chest drain already in situ and **IF IN ACCORDANCE WITH LOCAL POLICY** use two clamps to double clamp the chest catheter.
4. Attach the drain by pushing the fir-tree barb connector into the chest catheter, or the plain end of the tube onto the tapered Seldinger drain connector.
5. Ensure the white moisture absorbing material is placed towards the patient so that the venting discs are facing outwards.  
**DO NOT LAY DOWN SO FLUID CAN COVER THE VENT, THIS WILL PREVENT THE DRAIN FROM VENTING AIR**
6. Remove **BOTH** the clamps. The drain is now in use.
7. If the patient has a brisk air leak, open the blue valve at the top of the bag using a quarter twist and pulling motion. Place the red clip underneath the blue valve cap to prevent the valve being shut accidentally, and to indicate the valve is open.
8. Remember if the blue valve at the top of the bag is open, if the patient is draining fluid this may drain out of the valve.
9. Fit and adjust the belt by undoing the Velcro tag and threading through the belt loops on the valve protector. Then place, so that the bag sits comfortably on the patient's side. The belt size can be adjusted using the Velcro. Alternatively, the belt can be utilised as a shoulder strap.
10. To empty the bag use the valve at the bottom of the bag, remember to close the valve after use.
11. If at anytime during the use of the bag it becomes necessary to reconnect the patient to an underwater seal drain, apply two clamps to the chest catheter and disconnect the ambulatory bag from the catheter. Refit the drainage device with a fir-tree barb connector fitted. Remove the clamps; check the drainage device is working.
12. If the bag needs replacing with another ambulatory chest drain, apply two clamps to the chest catheter and follow steps 1 to 8 as above.
13. If the patient is draining large amounts of fluid, the drain may, **IN ACCORDANCE WITH LOCAL POLICY**, be connected to a larger fluid collection bag via the overnight bag connection.

**WARNINGS:**

**ONLY QUALIFIED PERSONNEL WHO ARE FAMILIAR WITH CHEST DRAIN CARE SHOULD ADMINISTER THE DRAINAGE BAG. FAILURE TO PRIME THE VALVE PRIOR TO USE, INAPPROPRIATE OR OCCLUSION OF THE VENT AREA MAY PREVENT THE BAG FROM FUNCTIONING CORRECTLY.**

**DO NOT DISCONNECT THE BAG WITHOUT CLAMPING THE CHEST CATHETER.**

**IF THE PATIENT HAS A BRISK AIR LEAK, OPEN THE BLUE VALVE BY PULLING THE BLUE CAP USING A QUARTER TWIST AND PULLING MOTION.**

**DO NOT LAY THE DRAIN FLAT; FLUID IN THE DRAIN MAY PREVENT AIR FROM VENTING.**

**FLUID CONTINUOUSLY IN CONTACT WITH THE VENT MATERIAL MAY CAUSE THE MATERIAL TO SATURATE AND LEAK FLUID. BLOCKED OR PARTIALLY BLOCKED CHEST DRAINAGE CATHETERS CAN RESULT IN COMPLICATIONS SUCH AS TENSION PNEUMOTHORAX.**

**IF THE PATIENT IS DRAINING A THICK EMPYEMA TYPE EXUDATE AND HAS AN AIRLEAK AT THE SAME TIME, MONITOR CLOSELY AND EXERCISE CAUTION IN THE USE OF THIS PRODUCT.**

**GRADUATIONS ARE ACCURATE TO +/- 10%**

**Disposal:** This device should be handled and disposed of in accordance with local hospital policy and with regard to all applicable regulations, including but without limitation to, those pertaining to human health & safety and care of the environment

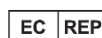
**Reorder Number: R54564**

This device is not manufactured with natural rubber latex

**ROCKET MEDICAL PLC**  
Sedling Road, Washington,  
England, NE38 9BZ  
www.rocketmedical.com

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CONTINUOUS USE SHOULD NOT EXCEED 28 DAYS  
DO NOT RESTERILISE

Unless opened or damaged, contents of package are sterile.  
Store at room temperature. Avoid prolonged exposure to elevated temperatures

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Copyright© 2013-2020.
**Rocket Medical GmbH**