R54410-00-DL INSTRUCTIONS FOR USE



Scope: These instructions cover the use of R54410-00-DL Rocket IPC Drainage Line.

Device Description: Contains 80mm drainage line with clamp and fir tree connector with valve for connection to the Rocket IPC catheter.

Indications: For connecting IPC Catheter to other drainage devices such as an underwaterseal chest drain or ambulatory bag.

Contraindications: Not for connection to any other type of indwelling pleural or peritoneal catheter unless used with the correct adapter provided by Rocket Medical.

Procedure: If you have a no clamp policy ignore steps 2 & 7.

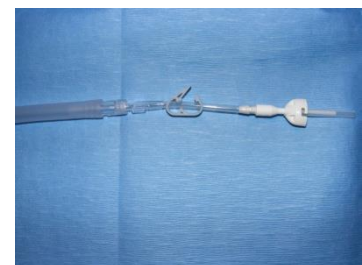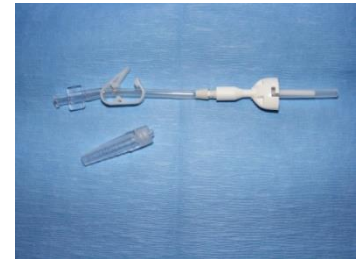1. If required attach the fir tree connector to the luer lock.
2. If required close the clamp.
3. Remove cap from IPC catheter by rotating anti-clockwise and pulling gently.
4. Clean valve with an alcohol wipe.
5. Attach drainage line to the collection or flushing device i.e. Underwater Seal Drain, Ambulatory Bag or syringe.
6. Remove the access tip cover, from the drainage line, keep sterile. Connect drainage line to IPC catheter by gently inserting the access tip into the valve. You will feel a little resistance while you do this, which is normal.
7. If you have closed the clamp, unclamp drainage line.
8. Monitor patient and if draining fluid - monitor the amount, colour and consistency of the fluid drained.
9. If flushing the catheter ensure you are using a suitable material such as an anti fibrinolytic.
10. When drainage complete, take the drainage line in one hand and the catheter valve in the other. Rotate the drainage line anti clockwise and slowly pull the access tip out of the valve in a smooth motion.
11. Use a **new alcohol wipe** to gently clean around the outside of the valve.
12. Take a **new** catheter valve cap and rotate it clockwise until it clicks into position.

Disposal: This device should be handled and disposed of in accordance with local hospital policy and with regard to all applicable regulations, including but without limitation to, those pertaining to human health & safety and care of the environment.

This device is not manufactured with natural rubber latex

STERILE EO

Rx ONLY

ROCKET MEDICAL PLC
Sedling Road, Washington,
England, NE38 9BZ
www.rocketmedical.com

EC REP

Rocket Medical GmbH
Am Rosengarten 48,
15566 Schöneiche,
Germany

DO NOT RESTERILISE

Unless opened or damaged, contents of package are sterile. Sterile unless package has been opened or damaged.

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