
Instructions for Use: Battery Removal

Scope: These instructions cover R53905 and R53905-HALO Rocket Pro-Lux Proctoscopes and derivatives.

Indications: This device should only be used by or under the supervision of trained personnel and in conjunction with current local clinical practice guidelines.

Disposal: This device should be handled and disposed of in accordance with local hospital policy and with regard to all applicable regulations, including but without limitation to, those pertaining to human health & safety and care of the environment.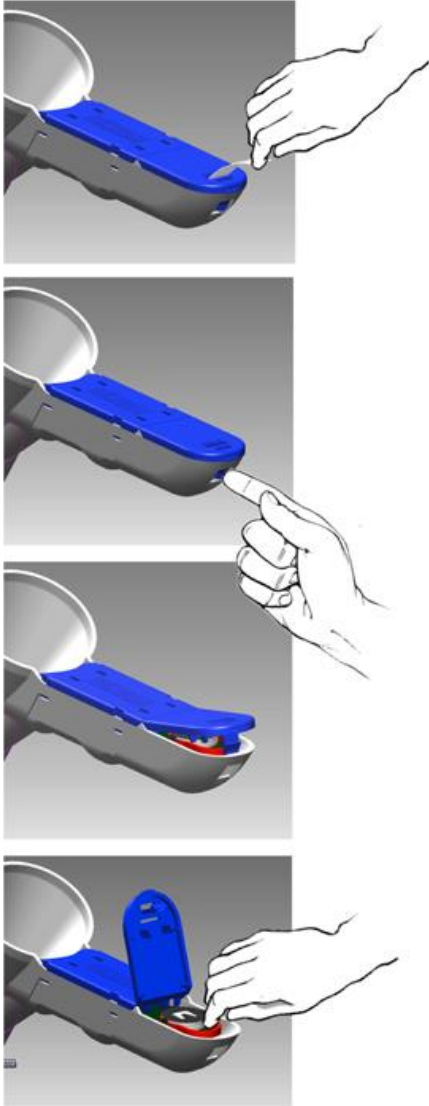

1. Remove tab to activate light source.
 2. Discard tab after removal.
 3. In accordance with local hospital policy, remove batteries before disposal.
 4. Push tab into handle to release battery cover.
 5. Lift cover to expose battery pack ready for removal.
 6. Pull the red tab shown to remove battery pack.
- HAZARD: Battery packs must NOT be allowed to come into contact with the batteries of other devices as they may short-out, leak fluid, overheat or explode.**
7. To ensure safety, place each battery pack into a separate cell of the disposal tray provided with each carton of Pro-Lux Proctoscopes.
 8. Dispose of the filled tray in accordance with local hospital policy.

This device is not manufactured with natural rubber latex

Rx ONLY

ROCKET MEDICAL PLC
Sedling Road, Washington,
England, NE38 9BZ
www.rocketmedical.com

EC REP

Rocket Medical GmbH
Am Rosengarten 48,
15566 Schöneiche,
Germany

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