

# Rocket Pleural Vent Occlusion Plug

## INSTRUCTIONS FOR USE



**Scope:** These instructions cover the Rocket R54565-CP Pleural Vent Occlusion Plug. This device should only be used by, or under the supervision of, appropriately trained personnel and in conjunction with current local clinical practice guidelines

**Indications:** For occluding the air flow path through the R54565 Rocket Pleural Vent

**Contraindications:** Not intended to allow air flow through the R54565 when in situ

### **Procedure:**

1. When the signal diaphragm remains in the down position and the patient is asymptomatic the pneumothorax may be resolved. This must be confirmed with imaging before removal of the Pleural Vent.
2. The valve system of the Pleural Vent can be occluded using the included spigot.
3. Remove the cap covering the valve.
4. Place the spigot of the occlusion plug through the valve and lock into place.
5. If airflow through the Pleural Vent is still required remove the occlusion plug.



**WARNING: IF PNEUMOTHORAX HAS NOT RESOLVED AND THE PLEURAVENT IS TO BE LEFT IN POSITION DO NOT LEAVE SPIGOT IN SITU. REMOVE AS SOON AS POSSIBLE.**

**Disposal:** This device should be handled and disposed of in accordance with local hospital policy and with regard to all applicable regulations, including but without limitation to, those pertaining to human health & safety and care of the environment.



This device is not manufactured with natural rubber latex



R<sub>x</sub> ONLY

STERILE EO



ROCKET MEDICAL PLC Sedling Road, Washington, England, NE38 9BZ [www.rocketmedical.com](http://www.rocketmedical.com)

EC REP

Rocket Medical GmbH  
Am Rosengarten 48,  
15566 Schöneiche,  
Germany

**DISCARD AFTER SINGLE USE - DO NOT RESTERILISE**  
Unless opened or damaged, contents of package are sterile.

