

Rocket


Pleural Vent Aspiration Line

INSTRUCTIONS FOR USE



Scope: These instructions cover Rocket R54565-AL Pleural Vent Aspiration Line.

This device should only be used by, or under the supervision of, appropriately trained personnel and in conjunction with current local clinical practice guidelines.

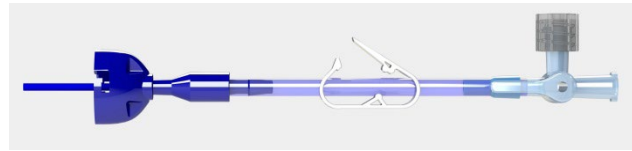

Indications: For aspirating air or applying thoracic suction through the R54565 Pleural Vent

Contraindications: Not intended for any other use except as indicated above.

Procedure:

1. Air can be aspirated through the pleural vent or thoracic suction can be applied to the patient using the Aspiration Line
2. Remove the cap covering the valve.
3. Place the spigot of the aspiration line through the valve and lock the cap in to place.
4. Connect a luer lock syringe, open the 3 way tap and aspirate air as required.
5. If wishing to connect to thoracic suction; connect the step connector to the 3 way tap and then connect to the vacuum source.
6. When finished with the Aspiration line, unlock and remove from the pleural vent. Cover valve with a new cap.

WARNING: IF ASPIRATION LINE 3 WAY TAP IS LEFT IN THE OFF POSITION THIS WILL PREVENT THE FLOW OF AIR THROUGH THE UNIT. IF THE PNEUMOTHORAX HAS NOT RESOLVED THE PATIENT IS AT RISK OF AN INDUCED TENSION PNEUMOTHORAX

DO NOT LEAVE ASPIRATION LINE IN SITU. REMOVE AS SOON AS POSSIBLE.

Disposal: This device should be handled and disposed of in accordance with local hospital policy and with regard to all applicable regulations, including but without limitation to, those pertaining to human health & safety and care of the environment.



This device is not manufactured with natural rubber latex



Rx ONLY



ROCKET MEDICAL PLC Sedling Road, Washington, England, NE38 9BZ www.rocketmedical.com



Rocket Medical GmbH
Am Rosengarten 48,
15566 Schöneiche.
Germany

DISCARD AFTER SINGLE USE - DO NOT RESTERILISE
Unless opened or damaged, contents of package are sterile.

