

Rocket IPC Dressing Pack


Scope: These instructions cover code R54400-00-00 and derivatives of Rocket IPC Dressing Pack.

Indications: For dressing of the wound and/or the catheter by patients fitted with Rocket IPC Catheter.


Contraindications: Not intended for any other use except the indicated above.

Preparation:

1. Use a clean, clear workspace, ideally a table or bench top. **Important:** Wash your hands with warm soapy water for at least 1 minute before you begin.
2. Remove the dressings from around the catheter site, if you notice anything unusual such as pain, redness, swelling or free fluid, finish this procedure first and then contact your doctor or nurse immediately.
3. Open your procedure pack and position the self-adhesive dressing to one side. Position the blue wrap on your clean workspace with the tape facing upwards. Carefully remove tape and unfold the blue wrapping making sure the contents remain in the middle of the sterile wrapping.
4. Put the gloves on. It is important that these remain sterile. Pick the first one up by the inside of wrist fold and place on the opposite hand, repeat this for the other hand. Your bare hand should not come into contact with the outer surface of the glove.
5. Clean around the catheter exit site with an alcohol pad (**fig.6**).
6. Place the foam catheter pad around the catheter (**fig.7**).
7. Wind the catheter carefully into loops and place over the foam (**fig.8**). Cover the catheter with the gauze pads (**fig.9**).

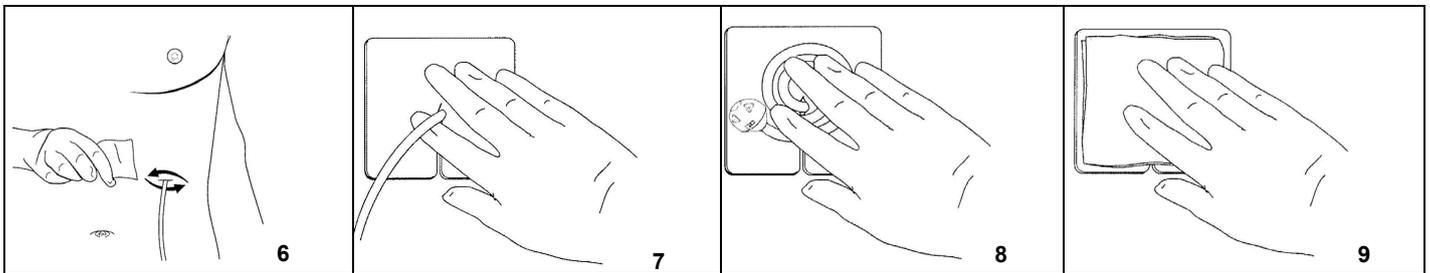

8. Take the wound dressing and peel away the paper backing from one side (**fig.10**).
9. Position the dressing centrally over the gauze pads and carefully press down (**fig.11**), then peel the other half of the dressing away pressing firmly all the time. Hold one of the coloured tabs on the dressing and pull the cover sheet away from the rest of the dressing, smooth and press until secure (**fig.12**).

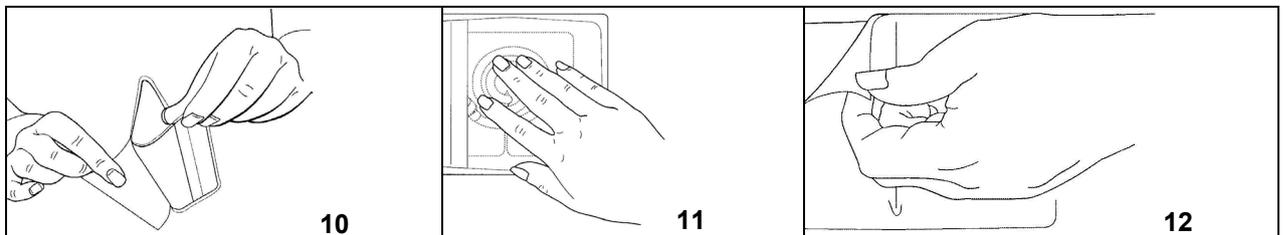

IMPORTANT: These instructions are intended as a guide, please refer to the patient pack that you should have received along with your training on the correct drainage procedure. If you are uncertain please contact your doctor or nurse.

Disposal: This device should be handled and disposed of in accordance with local hospital policy and with regard to all applicable regulations, including but without limitation to, those pertaining to human health & safety and care of the environment.



This device is not manufactured with natural rubber latex



Rx ONLY

STERILE EO



ROCKET MEDICAL PLC
Sedling Road, Washington,
England, NE38 9BZ
www.rocketmedical.com

EC REP

Rocket Medical GmbH
Am Rosengarten 48,
15566 Schöneiche.
Germany

DO NOT RESTERILISE

Unless opened or damaged, contents of package are sterile. Sterile unless package has been opened or damaged.

