

Rocket R54571 Filter for use with R54569 Digital Thoracic Suction Unit

INSTRUCTIONS FOR USE

Scope: These instructions cover the R54571 Filter.

Upon removal inspect the product to ensure no damage has occurred.

Device Description: The Rocket R54571 Filter contains a single patient use filter designed to prevent fluid ingress of the R54569 Mobile Suction Unit.

WARNING: Change the filter every 24 Hours

Indications: Single use filter for the prevention of fluid ingress of the R54569 Digital Mobile Suction Unit.

These products are intended for use by physicians trained and experienced in diagnostic and interventional techniques

Contraindications: Not intended to work in conjunction with any other digital suction unit except R54569.

Procedure:

1. Place the filter unit on the suction port of the chest drain bottle.
2. Ensure the location peg is in the location port on the chest drain bottle and the filter unit is securely in place.
3. Place and secure the suction unit to the chest drain bottle, and set suction level for use as per IFU for the R54569.

WARNING: Change the filter every 24 Hours

CAUTION: If the filter becomes contaminated with fluid, this may restrict the performance of the suction unit and should be changed immediately.

CAUTION: If there is any evidence that the pump may have become contaminated with fluid during use or the filter has failed in any other way, it must be removed from service and returned for examination immediately.

WARNING: When the filter is being removed from the suction port after use, the red seal moulding may on occasion become separated from the filter unit itself. If this is to happen, remove the moulding from the filter port and dispose of with the filter unit body.



Disposal: This device should be handled and disposed of in accordance with local hospital policy and with regard to all applicable regulations, including but without limitation to, those pertaining to human health & safety and care of the environment.



This device is not manufactured with natural rubber latex



Rx ONLY

STERILE EO



ROCKET MEDICAL PLC
Sedling Road, Washington,
England, NE38 9BZ
www.rocketmedical.com

EC REP

Rocket Medical GmbH
Am Rosengarten 48,
15566 Schöneiche,
Germany

CONTINUOUS USE SHOULD NOT EXCEED 7 DAYS

For Single Use Only. Do not reuse on another person, reprocess or re-sterilise as doing so may compromise the structural integrity of the device, leading to device failure; potentially the cause of serious harm to patients and users. Reuse, reprocessing or re-sterilisation may also result in serious harm to patients and users from cross-contamination and infection with transmissible diseases. **Unless opened or damaged, contents of package are sterile.**



1639

ZDOCK286

2021-04-28

Rev.05

Copyright© 2015-21.

ROCKET MEDICAL PLC

All Rights Reserved