Rocket Medical IPC Bottle/Bag Adapter

ZDOCK289 Rev 07

Language	Page
EN.....	2
CS.....	3
DA.....	4
DE.....	5
ES.....	6
FR.....	7
IT.....	8
NL.....	9
PL.....	10
PT.....	11
AR.....	12
TR.....	13

# Rocket Medical IPC Bottle/Bag Adapter

## Important Information Read Before Draining


**Indication:** For the adaption of the Rocket Medical IPC drainage bottle or bag for connection with the BD Pleurx™ catheter.

**Contraindication:** Not for connection to any other drainage equipment or catheter except those indicated above.

**PLEASE NOTE:** If you have a Rocket Medical IPC catheter please follow the other instructions for use included in the packaging with the Rocket Medical drainage bottles and bags.

If your IPC drainage catheter has a valve similar to the one shown in the image to the right (Diag. A), you can use the bottle/bag adapter to connect Rocket Bottles and Bags to your catheter valve for draining.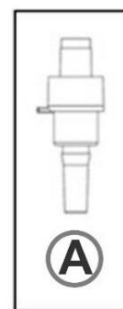

**TO PROCEED WITH CONNECTION OF THE VALVE ADAPTOR TO THE ROCKET BOTTLE / BAG, PLEASE READ AND FOLLOW THE INSTRUCTIONS BELOW:**


Follow the separate bottle or bag instructions for use until **Stage 7**. At this point open the IPC adapter pack in to the middle of the blue wrap.

1. Detach the tubing and unravel the tubing from the bottle or bag and remove access tip protector. 1
2. Take the adapter and slide over the access tip. **Image 1.**
3. Push the adapter and the drainage line together so it clicks together and becomes secure. **Image 2.** 2
4. Remove your catheter cap and gently clean around the outside of the catheter valve with an alcohol wipe.
5. Take the drainage line of the bottle and gently insert the access tip into the valve. You will feel a little resistance while doing this, which is normal. **Image 3.** 3
6. Lock together as shown. **Image 4.**

When drainage has been completed, please resume from instruction **Stage 14** in the instructions for use (provided with your bottles or bags). 4

Use the cap supplied with the adapter. Rotate the cap to lock it into position. **Image 5.**

Finally, if required apply the dressing as directed in the instructions for use. 5

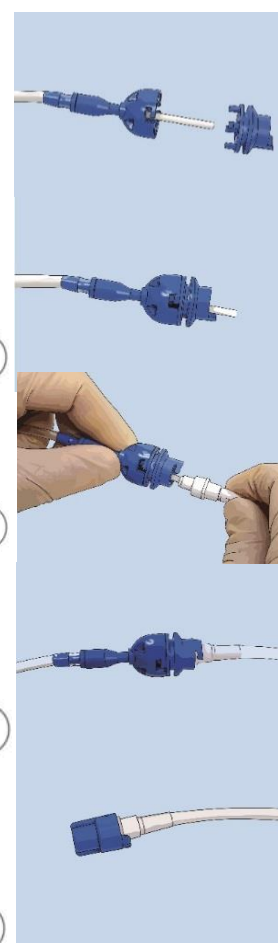This device is not manufactured with natural rubber latex

ROCKET MEDICAL PLC  
Sedling Road, Washington,  
England, NE38 9BZ  
[www.rocketmedical.com](http://www.rocketmedical.com)

Rocket Medical GmbH  
Am Rosengarten 48,  
15566 Schöneiche,  
Germany

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