

# Haemorrhoidal Injection Sets

## Instructions for Use

**Scope:** These instructions cover R56210 Haemorrhoidal Injection Set-Straight and R56211 Haemorrhoidal Injection Set-Angled.

**Indications:** For the administration of oily Phenol BP or other sclerosing agents for the treatment of internal haemorrhoids. This device should only be used by, or under the supervision of, appropriately trained personnel and in conjunction with current national and local clinical practice guidelines

**Contraindications:** The device is not for any purpose other than the treatment of internal haemorrhoids. It should not be used on patients with:


- Active rectal inflammatory disease
- Immunosuppression

### Procedure:

1. Remove the syringe from the packaging and attach the filling kwill.
2. Draw up the oily Phenol BP or sclerosing agent.
3. Remove the filling kwill from the syringe and attach the needle.
4. Clean the anus with an antiseptic solution according to your hospital policy.
5. Lubricate an appropriately sized ano-proctoscope, such as the Rocket Medical Pro-Lux™ Proctoscope (product code R53905), with water-soluble lubricant.
6. Using the ano-proctoscope, locate the haemorrhoid and the injection site, ensuring that that this is above the dentate line.
7. Adminster the scleroscant into the submucosal layer at the base of the haemorrhoid. The volume injected should be made according to the size of the haemorrhoid and in line with the manufacturer's guidelines.
8. Repeat the procedure if required, taking account of maximum scleroscant administration guidelines in order to prevent toxicity.

Pain and or bleeding may occur following treatment; pain medication may be required. Serious complications of sclerotherapy are rare and include embolism, allergic reaction to scleroscant, rectal perforation, rectourethral fistula, sepsis, abscess and abdominal compartment syndrome. Urinary complications may rarely occur in male patients.

**Disposal:** This device should be handled and disposed of in accordance with local hospital policy and with regard to all applicable regulations, including but without limitation to, those pertaining to human health & safety and care of the environment.

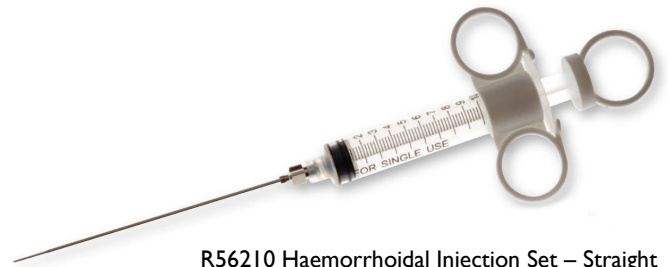


R56210 Haemorrhoidal Injection Set – Straight



This device is not manufactured with natural rubber latex



**ROCKET MEDICAL PLC** Sedling Road, Washington, England, NE38 9BZ [www.rocketmedical.com](http://www.rocketmedical.com)

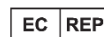

**Rocket Medical GmbH**  
Am Rosengarten 48,  
15566 Schöneiche.  
Germany

